

Model CH-1

comfortHer™

Post-Surgery Bra Strap Pad


Using the comfortHer™ Post-Surgery Bra Strap Pad

The comfortHer™ Post-Surgery Bra Strap Pad can be adjusted to fit any bra strap using the two Velcro strips on the front of the device. To wear:

1. Open the two Velcro strips on the front of the pad.
2. With the bra in place, first slide the open Velcro strips and then the comfortHer pad under the bra strap so that it is positioned over the surgical site.
3. Close both Velcro strips over the bra strap, making sure that the ends are firmly secured to the opposing Velcro strips after crossing the bra strap.
4. Adjust as necessary to provide the maximum comfort over the surgical site.

Care Instructions

Close the Velcro strips before washing. Hand wash with mild soap in warm water only. Rinse well and air dry. DO NOT machine wash, tumble dry, dry clean, or use bleach or fabric softeners.

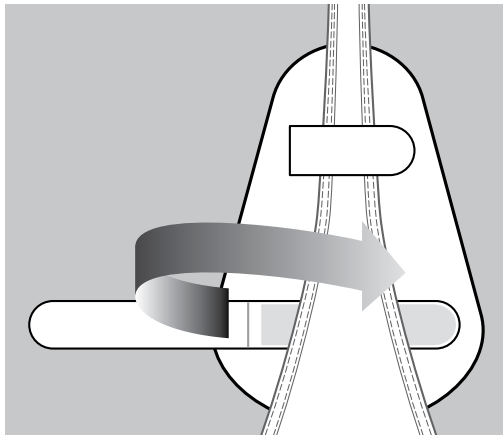

The comfortHer™ Post-Surgery Bra Strap Pad is guaranteed to be of the highest quality construction. If you should receive a defective product, please contact Pressure Products at (800) 600-4973 for a replacement or full refund.

WARNING: If discomfort or pain results, discontinue use and contact a physician.

comfortHer is a trademark of Pressure products, Inc. © Copyright 2007, Pressure Products, Inc. All rights reserved. Made in China. Printed in the U.S.A.

Pressure Products, Inc. Customer Service
1861 N. Gaffey Street Suite B San Pedro, CA 90731
(800) 600-4973 (310) 547-4973 Fax (310) 547-4760
www.pressure-products.com

 **Pressure™**
Products
ACCESSING THE HEART OF INNOVATION