Pacing Lead Stabilizer

For use with SafeSheath Introducer

Stabilisateur d'électrode de stimulation
Stabilisator für die Zuleitung des Schrittmachers
Estabilizador para electrodo de marcapaso
Lo stabilizzatore per conduttori
Estabilizador de guia de marca-passo
Stabilisator voor de pacingelektrode
Pacing Leder Stabilisatoren
Stabilisator för pacemakersladdar
Tahdistusjohtimen vakainta
Stabilisator för pacemakersladdar
 Σταθεροποιητής Αγωγού Βηματοδότησης
 ペーシングリード安定装置





Read instructions for use prior to use

Single use only

STERILE EO

Contents are sterile if package is unopened and without damage. Sterilized with Ethylene Oxide.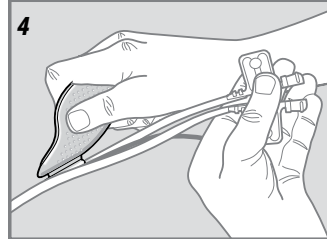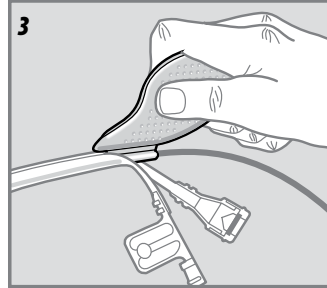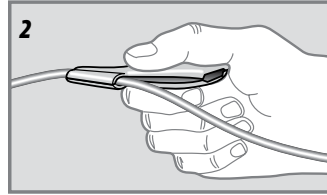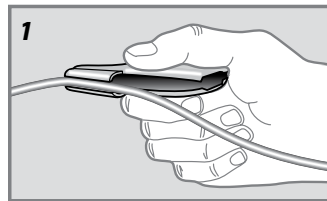

Instructions for Use

To aid in removal of a splittable introducer after it has been used to place a pacemaker lead or catheter.

All PLS materials are biocompatible, latex-free and PVC-free.

Precautions

Do not alter this device in any way.

Federal law (U.S.A.) restricts this device to sale by or on the order of a physician.

As specified in the following chart, use the appropriate size PLS for the desired pacing lead and introducer sheath.

PLS Lead Size Range	Smallest Introducer Sheath
3.6/6.0 F 1.2/2.0 mm	7F
3.6/8.0 F 1.2/2.7 mm	9F

Suggested directions for use

Note: The Pacing Lead Stabilizer (PLS) is to be used after pacing lead or catheter is in the desired location and the sheath introducer is ready to be removed.

1. Open package and place contents on sterile field.
2. Separate the handles of the sheath and peel the sheath approximately 1 cm.
3. Place the nose of the PLS around the pacing lead or catheter close to the proximal end of the sheath. The pacing lead or catheter will sit between two sides of the PLS. See Figure 1.
4. Compress the PLS around the pacing lead and slide towards the sheath. See Figure 2.
5. Keep advancing the PLS until the nose engages the inside diameter of the sheath and the tapered section is aligned with the pre-weakened section of the introducer sheath. See Figure 3.
6. To remove the sheath, fully compress the PLS to hold the lead. Hold both handles of the sheath and slowly retract the sheath from vessel past the PLS. The stabilizer will assist in separating the sheath tubing longitudinally while maintaining lead position. See Figure 4.

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Pressure Products, Inc.