H.Lee SafeSheath® Sealing Adapter

For use with Medtronic Attain™ Access Delivery System

STERILE EO

For U.S.-California Only.
Proposition 65: a State of California voter initiative, requires the following notice:

WARNING: This product and its packaging have been sterilized with ethylene oxide. This packaging may expose you to ethylene oxide, a chemical known to the state of California to cause cancer or birth defects or other reproductive harm.

Pressure Products

Customer Service
1861 N. Gaffey Street, Suite B
San Pedro, CA 90731 - USA
Tel +1-310-547-4973
Fax +1-310-547-4760
www.pressure-products.com

Merit Medical Systems, Inc.
65 Great Valley Parkway
Malvern, PA 19355 - USA
Tel +1-801-253-1600
USA Customer Service:
Tel +1-800-356-3748

EC REP


Merit Medical Ireland, Ltd
Parkmore Business Park West
Galway, Ireland
European Customer Service
Tel +31 43 358 82 22

CE

0344

**Pressure®
Products**
ACCESSING THE HEART OF INNOVATION

CAUTION: Federal (U.S.A.) Law restricts this device to sale by or on the order of a physician.

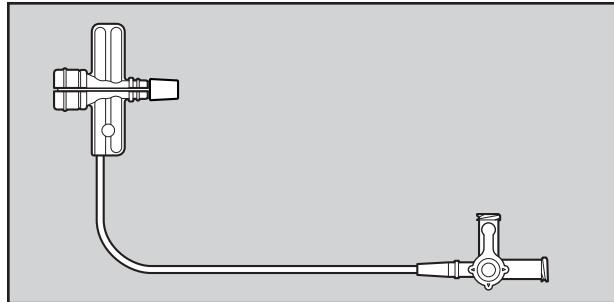Technical Manual

en Directions for Use	2
2 تعلیمات الاستخدام	ar
cs Pokyny k použití	2
da Brugsanvisning	2
de Gebrauchsanleitung	3
el Οδηγίες χρήσης	3
es Instrucciones para el uso	3
et Kasutusjuhised	4
fi Käyttöohjeet	4
fr Mode d'emploi	4
hr Uputeza upotrebu	4
hu Használati utasítás	5
it Istruzioni per l'uso	5
ja 使用説明書	5
kk Пәйдалану жөніндегі нұсқаулар	5
ko 사용 지침	6
lt Naudojimo instrukcijos	6
nl Gebruiksaanwijzing	6
no Bruksanvisning	7
pl Sposób używania	7
pt Instruções de Utilização	7
ru Инструкции по использованию	7
sk Pokyny na použitie	8
sr Упутства за употребу	8
sv Bruksanvisning	8
tr Kullanım Talimatları	8
zh 使用说明	9

The H. Lee SafeSheath® hemostatic splittable sheath is protected by Patent Insurance. USA Patent 8,048,061. Canadian Patent 2,493,189. European Patent EP1545677. Japanese Patent 4705779. Other Worldwide Patents pending. SafeSheath is a registered trademark of Pressure Products Medical Supplies, Inc. Attain is a trademark of Medtronic, Inc.

en-Directions for Use

This device is intended for one time use only. Read instructions prior to use.

Indications

For the introduction of various types of pacing leads and catheters.

Warnings





Infusion through the sideport can be done only after all air is removed from the unit.

Precautions

- Do not alter this device in any way.
- **Single Use Devices:** This single-use product is not designed or validated to be reused. Reuse may cause a risk of cross-contamination, affect the measurement accuracy, system performance, or cause a malfunction as a result of the product being physically damaged due to cleaning, disinfection, re-sterilization, or reuse.
- Aspiration and saline flushing of the SafeSheath® Sealing Adapter should be performed to help minimize the potential for air embolism and clot formation.
- Dilators, catheters, and pacing leads should be removed slowly from the SafeSheath® Sealing Adapter. Rapid removal may damage the valve members resulting in blood flow through the valve.
- When injecting or aspirating through the SafeSheath® Sealing Adapter, use the sideport only.

Use Sterile Technique A suggested procedure:

1. Peel open the package and place the contents on the sterile field.
2. Attach the SafeSheath® Sealing Adapter to the proximal end of the Attain™ Guide Catheter by inserting the SafeSheath® Sealing Adapter into the Attain™ Guide Catheter hub until the top of the elastic plug is flush with top of the hub. (see figure 1)
3. Aspirate all air from the sheath valve assembly by using a syringe connected to the sideport. Flush the SafeSheath® Sealing Adapter through the sideport. If the SafeSheath® Sealing Adapter is to remain in place during lead positioning and testing, flush the SafeSheath® Sealing Adapter via the sideport periodically with saline as advised.
4. The hemostasis valve of the SafeSheath® Sealing Adapter will reduce the loss of blood and the inadvertent aspiration of air through the sheath.
5. Introduce the pacemaker lead or catheter through the hemostasis valve of the SafeSheath® Sealing Adapter and advance it into position.
6. Flush the SafeSheath® Sealing Adapter with 5cc of saline immediately before peeling the SafeSheath® Sealing Adapter away in order to minimize postbleeding.
7. With the SafeSheath® Sealing Adapter positioned within the Attain™ Guide Catheter hub, sharply snap the tabs of the SafeSheath® Sealing Adapter in a plane perpendicular to the long axis of the sheath to separate the valve. (see figure 2)
8. Holding the Attain™ Guide Catheter hub flange with one hand and the SafeSheath® Sealing Adapter with the other, withdraw the SafeSheath® Sealing Adapter from the Attain™ Guide Catheter hub while maintaining the lead position. (see figure 3)
9. Peel the SafeSheath® Sealing Adapter apart to completely separate the SafeSheath® Sealing Adapter from the pacing lead or catheter. (see figure 4)

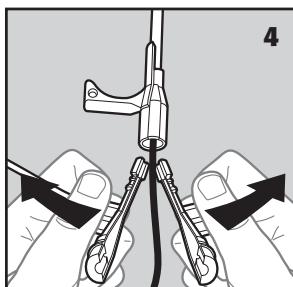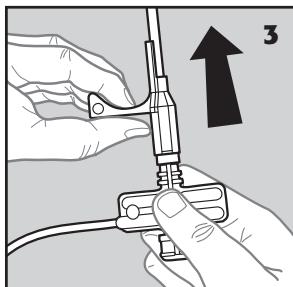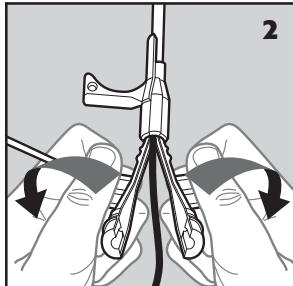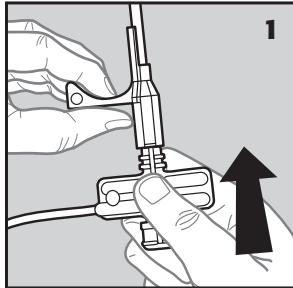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