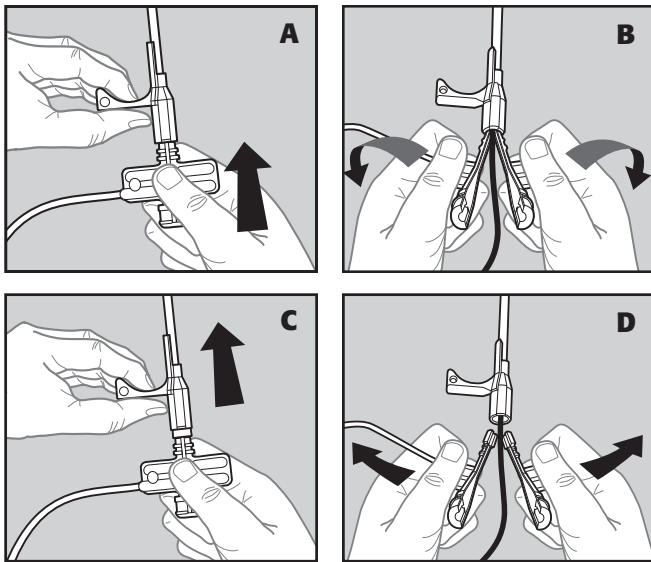
# H. Lee SafeSheath® Sealing Adapter

For use with Medtronic Attain™ Access Delivery System

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**Read instructions for use prior to use** **Single use only** **STERILE EO** **Contents are sterile if package is unopened and without damage. Sterilized with Ethylene Oxide.**

## Directions for Use

This device is intended for one time use only. Read instructions prior to use.

**Indications:** For the introduction of various types of pacing leads and catheters.

## Precautions

1. Do not alter this device in any way.
2. Aspiration and saline flushing of the SafeSheath® Sealing Adapter should be performed to help minimize the potential for air embolism and clot formation.
3. Dilators, catheters, and pacing leads should be removed slowly from the SafeSheath® Sealing Adapter. Rapid removal may damage the valve members resulting in blood flow through the valve.
4. When injecting or aspirating through the SafeSheath® Sealing Adapter, use the sideport only.

## Use Sterile Technique A suggested procedure:

1. Peel open the package and place the contents on the sterile field.
2. Attach the SafeSheath® Sealing Adapter to the proximal end of the Attain™ Guide Catheter by inserting the SafeSheath® Sealing Adapter into the Attain™ Guide Catheter hub until the top of the elastic plug is flush with top of the hub. (see figure A)
3. Aspirate all air from the sheath valve assembly by using a syringe connected to the sideport. Flush the SafeSheath® Sealing Adapter through the sideport. If the SafeSheath® Sealing Adapter is to remain in place during lead positioning and testing, flushing the SafeSheath® Sealing Adapter via the sideport periodically with saline is advised.
4. The hemostasis valve of the SafeSheath® Sealing Adapter will reduce the loss of blood and the inadvertent aspiration of air through the sheath.
5. Introduce the pacemaker lead or catheter through the hemostasis valve of the SafeSheath® Sealing Adapter and advance it into position.
6. Flush the SafeSheath® Sealing Adapter with 5cc of saline immediately before peeling the SafeSheath® Sealing Adapter away in order to minimize backbleeding.
7. With the SafeSheath® Sealing Adapter positioned within the Attain™ Guide Catheter hub, sharply snap the tabs of the SafeSheath® Sealing Adapter in a plane perpendicular to the long axis of the sheath to separate the valve. (see figure B)
8. Holding the Attain™ Guide Catheter hub flange with one hand and the SafeSheath® Sealing Adapter with the other, withdraw the SafeSheath® Sealing Adapter from the Attain™ Guide Catheter hub while maintaining the lead position. (see figure C)
9. Peel the SafeSheath® Sealing Adapter apart to completely separate the SafeSheath® Sealing Adapter from the pacing lead or catheter. (see figure D)

## Warnings

Infusion through the sideport can be done only after all air is removed from the unit.

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