

SiteLIGHT™

Surgical Light



See instructions for use and storage.



Single use only



CAUTION: Federal (USA) law restricts this device to sale by or on order of a physician.

STERILE | EO

Contents of this package have been ethylene oxide sterilized.


For U.S.-California Only.

Proposition 65, a State of California voter initiative, requires the following notice:

WARNING: This product and its packaging have been sterilized with ethylene oxide. This packaging may expose you to ethylene oxide, a chemical known to the state of California to cause cancer or birth defects or other reproductive harm.



This product is light sensitive. Do not use if stored outside protective outer carton. Store in a cool, dark, and dry place.

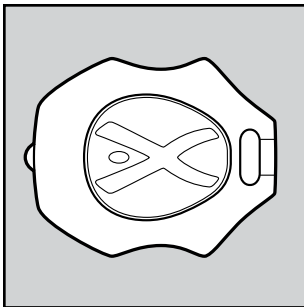

INTENSE LIGHT: Lamp is bright. Do not stare at the lamp when in use. May be harmful to eyes.

Distributed by

Pressure Products, Inc.

Customer Service
1861 N. Gaffey Street Suite B
San Pedro, CA 90731 USA
1.310.547.4973
1.800.600.4973
Fax 1.310.547.4760
www.pressure-products.com



Manufacturer

Oscor Inc.

3816 DeSoto Boulevard
Palm Harbor, FL 34683 USA
1.727.937.2511
Fax 1.727.934.9835
www.oscor.com



Patents Pending.
SiteLight is a trademark of Pressure Products, Inc.

Directions for Use

This device is intended for one time use only. Read instructions prior to use.

Indications for Use

Indicated for use in procedures where additional light at the surgical site is desired.

Warnings

- For one use only, do not re-sterilize.
- Intense Light-LED is bright. Do not stare at the LED when in use. May be harmful to eyes.

Precautions

- Store in a dry, dark, cool place.
- Exposure to temperatures above 54° C (130° F) may damage the product.
- Do not use if package is open or damaged.
- If the desired attachment area is wet, wipe dry prior to adhering the surgical light.
- May not adhere to surgical gloves with external coatings.

Use Sterile Technique *A suggested procedure:*

1. Peel open package and place SiteLight™ Surgical Light on sterile field.
2. When ready to use, remove adhesive protective film and discard.
3. Attach light to desired location of glove, drape, surgical tool, etc. by firmly placing the adhesive disk onto the desired area.
4. Turn on light by firmly pressing the top-mounted pushbutton on/off switch (do not hold switch down).
5. To turn off light, firmly press the top-mounted pushbutton on/off switch (do not hold switch down).
6. The light may be repositioned by carefully prying the light from the attached surface and repeating step 3.