
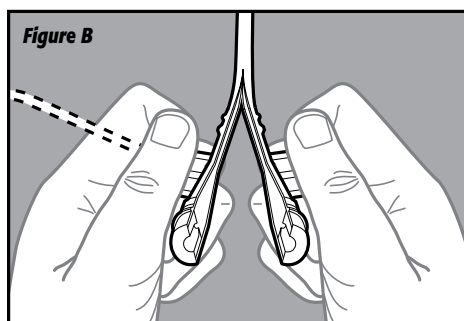
H. Lee **SafeSheath[®] D-Pro**


Hemostatic, Peel-away, Dialysis Introducer for Venous Access
Introduceur hémostatique détachable de dialyse pour accès veineux
Introductor hemostático para diálisis, pelable (sistema peel-away) para el acceso venoso
Introduttore rimovibile emostatico di dialisi per accesso venoso
Introduitor hemostático de diálise para puxar para acesso venoso
Hemostatische, peel-away, dialyse-introducer voor veneuze toegang
Hæmostatisk, afskrællelig dialyseintroducer til veneadgang
Hemostatisk, avskalbar dialysintroducer för venåtkomst
Hemostaattinen, irrotettava, dialyysin sisäänviejä laskimokäyttöä varten
Blodstillende, avtrekkbar, dialyse-innføringsenhet for venøs tilgang
Αιμοστατικός, αποχωριζόμενος εισαγωγέας αιμοκάθαρσης για φλεβική πρόσβαση
静脈アクセス用止血ピールアウェイ透析イントロデューサ

Directions for Use	2
Mode d'emploi	3
Gebrauchsanleitungen	4
Indicaciones para el uso	6
Istruzioni per l'uso	7
Instruções de uso	8
Gebruiksaanwijzing	9
Brugsanvisning	10
Bruksanvisning	11
Käyttöohjeet	12
Bruksanvisninger	13
Οδηγίες Χρήσης	14
使用说明	15

Figures

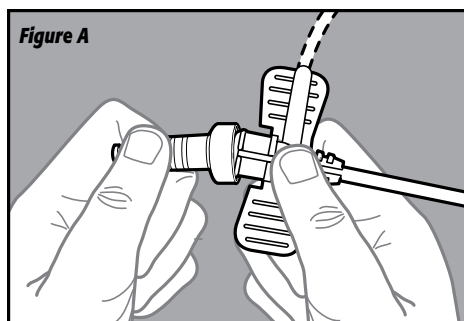 **Size**/Taille/Größe/Tamaño/Misura/Tamanho/Maat/
Størrelse/Storlek/Størrelse/Koko/Mέγεθος/サイズ

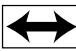 **Length**/Longueur/Länge/Longitud/Lunghezza/
Comprimento/Lengte/Længde/Längd/lengde/
Pituus/Mήκος/長さ

Directions for Use

This device is intended for one time use only. Read instructions prior to use.

NOTE: Text highlighted in gray refers to introducer models with side port only.

Indications

For the introduction of various types of catheters.

Warnings

- **This product is sensitive to light. Do not use if stored outside the protective outer carton. Store in a cool, dark, and dry place.**
- Infusion through the side port can be done only after all air is removed from the unit.

Precautions

- Do not alter this device in any way.
- Federal law (U.S.A) restricts this device to sale by or on the order of a physician.
- Aspiration and saline flushing of the sheath, dilator, and valve should be performed to help minimize the potential for air embolism and clot formation.
- Indwelling introducer sheaths should be internally supported by a catheter, electrode, or dilator.
- Dilators and catheters should be removed slowly from the sheath. Rapid removal may damage the valve members resulting in blood flow through the valve. Never advance or withdraw guide wire or sheath when resistance is met. Determine cause by fluoroscopy and take remedial action.
- When injecting or aspirating through the sheath, use the side port only.

Use Sterile Technique *A suggested procedure:*

1. Peel open package and place contents on sterile field.
2. Prep skin and drape in area of anticipated veni-puncture as desired.
3. Distend the subclavian vein or internal jugular vein. The subclavian vein is difficult to locate unless it is distended by raising the patients legs to a 45° angle or by using the Trendelenburg position. The vein will be much easier to locate if the patient is well hydrated.
4. Insert needle into vessel. The needle position should be verified by observing venous blood return. (Puncture the vein as medially as possible, by entering just lateral to the ligament which joins the clavicle and the first rib.)

Mode d'emploi

- The angle of the needle should be adjusted depending on the Patient's build: shallow in a thin person, deeper in a heavyset person. Use an 18 gauge needle, 7cm (2-3/4 in.) long.
- Aspirate the puncture needle using the 12cc syringe.
- Remove the syringe and insert soft tip of guide wire through the introducer needle into the vessel. Advance guide wire guide to required depth. Leave an appropriate amount of guide wire exposed.

At no time should the guide wire be advanced or withdrawn when resistance is met.

Determine the cause of resistance before proceeding. Fluoroscopic verification of the guide wire's entrance into the superior vena cava and right atrium is suggested.

- Hold guide wire in place and remove introducer needle. Do not withdraw the guide wire back into the cannula as this may result in separation of the guide wire. The cannula should be removed first.
- Insert vessel dilator into sheath until the dilator cap folds over valve housing and secures the dilator onto sheath assembly.
- Thread the dilator/sheath assembly over the guide wire.
- Advance the dilator and sheath together with a twisting motion over the guide wire and into the vessel. Fluoroscopic observation may be advisable. Attaching a clamp or hemostat to the proximal end of the guide wire will prevent inadvertently advancing the guide wire entirely into the patient.
- Once assembly is fully introduced into the venous system, separate the dilator cap from the sheath valve housing by rocking the dilator cap off the hub. (see figure A)
- Slowly retract the guide wire and dilator, leaving the sheath in position. The hemostasis valve will reduce the loss of blood and the inadvertent aspiration of air through the sheath.

14. Aspirate all air from the sheath valve assembly by using a syringe connected to the side port. Flush the introducer through the side port. If the introducer is to remain in during subsequent procedures, flushing the introducer via the side port periodically with saline is advised.

- Introduce catheter through the hemostasis valve/sheath and advance it into position.
- Flush sheath with 5cc of saline immediately before peeling sheath away in order to minimize backbleeding.
- Withdraw sheath and valve over the catheter and from the vessel, while keeping the lead in place.
- Sharply snap the tabs of valve housing in a plane perpendicular to the long axis of the sheath to split the valve and peel sheath apart while withdrawing from the vessel. (see figure B)

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Manufactured by

Thomas Medical Products, Inc.

65 Great Valley Parkway
Malvern, PA 19355 USA
1.610.296.3000 Fax 1.610.296.4591
www.thomas-medical.com

Distributed by

Pressure Products, Inc.

Customer Service
1861 N. Gaffey St. Suite B
San Pedro, CA 90731 USA
1.310.547.4973
1.800.600.4973
Fax 1.310.547.4760
www.pressure-products.com

Authorized European Representative

Vital Signs, Ltd.

13-14 Eldon Way
Lineside Industrial Estate
Littlehampton, West Sussex, UK

The H. Lee SafeSheath® splittable hemostatic sheaths are protected by Patent Insurance through 2011. USA Patents 5,125,904, 5,312,355 and 5,409,463. Canadian Patent 2,112,394. European Patent EP0593685.

SafeSheath is a registered trademark of Pressure Products, Inc.