





## Seat Belt **Relief** for Implant Patients



Support and comfort during everyday travel.  
SoftTouch™ is excellent for patients with

- Pacemakers
- Defibrillators
- Mastectomies
- Chemotherapy ports
- Other medical conditions

where seat belt irritation is a problem.

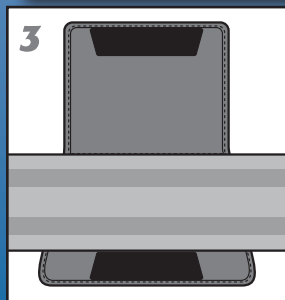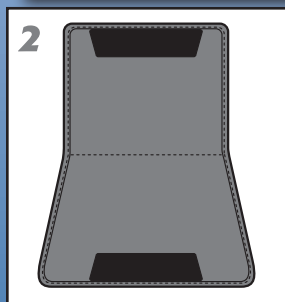

The patented **SoftTouch** seat belt pad is designed to provide **comfort** and a level of **protection** to those with pacemakers, defibrillators, chemotherapy ports, or other medical conditions while wearing a seat belt. The recessed underbelly, along with the open "C" shape design, enables the medical device to sit in a free air space without direct contact with the seat belt itself. This eliminates seat belt irritation over the area where the medical device is located, providing comfort.

### Instructions

1. Before using, first attach the appropriate sticker to the front of the SoftTouch (the side with the logo) identifying your implanted device for EMT alert (Figure 1).
2. Open the flap of the SoftTouch device (Figure 2).
3. Place the device between the seat belt and your clothes. Make sure the open end of the SoftTouch is pointing toward your shoulder, and the recessed underbelly is over your medical implant.
4. Place the seat belt between the Velcro, on top of the SoftTouch pad and flap (Figure 3).
5. Close the flap, attaching the Velcro of the flap and SoftTouch pad (Figure 4).

**NOTE:** If needed, you can place additional Velcro strips on the seat belt and SoftTouch pad to minimize excess movement.

U.S.A. Patent # 6557895.  
SoftTouch is a trademark of  
Pressure Products, Inc.  
© 2006, Pressure Products, Inc.  
All Rights Reserved.

 **Pressure  
Products**  
ACCESSING THE HEART OF INNOVATION

Distributed by

**Pressure Products, Inc.** Customer Service  
1861 N. Gaffey Street Suite B San Pedro, CA 90731 USA  
1.310.547.4973 Fax 1.310.547.4760  
[www.softtouchinfo.com](http://www.softtouchinfo.com)