
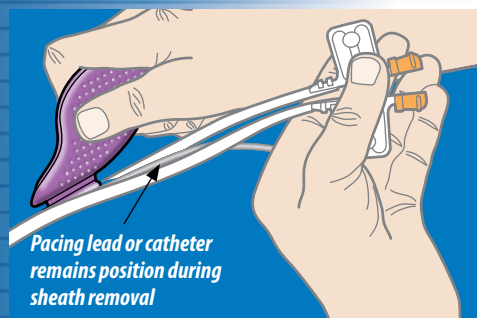
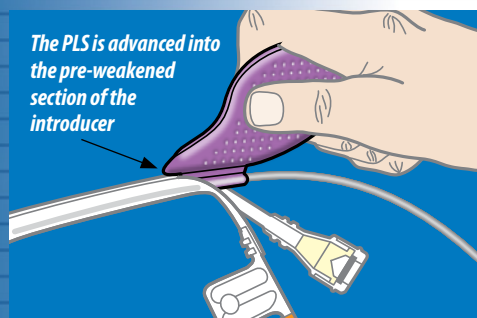
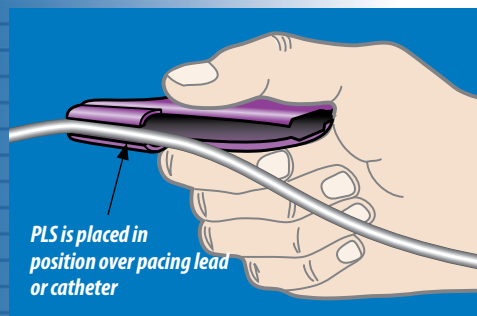


Pacing Lead Stabilizer

An aid for retaining lead position during removal of a splittable introducer, such as SafeSheath® or SafeSheath® CSG®.

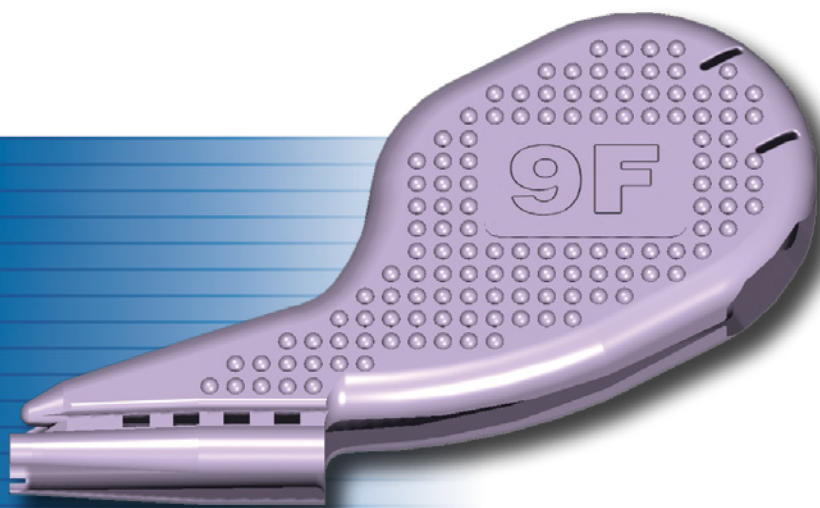

Introducing the **Pacing Lead Stabilizer (PLS)** from Pressure Products, the newest aid in the removal of a splittable introducer, such as SafeSheath or SafeSheath CSG, after placing a pacemaker lead or catheter.

After lead placement and threshold testing has been completed, the Pacing Lead Stabilizer is used to assist in sheath removal. When the valve has been manually split, securing the lead with the PLS in one hand allows for retraction of the scored sheath from the vein with the other hand, splitting the sheath in the process. The need for an assistant or inadvertent lead dislodgement is minimized.

Pacing Lead Stabilizer is part of a complete line of implantable device therapy products and accessories from Pressure Products.

Features:

- Retains lead position while introducer is retracted and split
- Lessens the need for an assistant during sheath removal
- Inadvertant lead dislodgement is minimized
- Can handle lead sizes between 4.3 to 8.0F and introducers from 9F and larger

Model	Description	Lead Size Range	Smallest Introducer Sheath	No. per Box
PLS-09	Pacing Lead Stabilizer	4.3 - 8.0F / 1.4 - 2.7mm	9F	5

P
R
O
D
U
C
T
I
N
F
O
R
M
A
T
I
O
N